

STANDARD SET

ADVANCED KITS



SUBSTANCE		STEP 1: WHITE TEST MARQUIS	STEP 2: RED TEST MECKE	STEP 3: BLUE TEST MANDELIN	STEP 4: TURQUOISE & GREEN TESTS FOLIN	STEP 5: ORANGE TEST FROEHDE	STEP 6: PINK TEST LIEBERMANN
		Reacts immediately. Good for weeding out bath salts and amphetamines.	Reacts immediately. Can detect DXM when no MDMA is present. Can also help identify some of the common cathinones.	Reacts immediately. Can detect PMA/PMMA when no MDMA is present. Also good for testing ketamine, MXE.	2-stage test. Can take up to 2 minutes to react after 2nd drop. Differentiates MDMA from MDA & MDE. Good for detecting piperazines.	Reacts immediately. Can more distinctly differentiate 5-APB or 6-APB from MDMA.	Reacts immediately. Can help differentiate which cathinone may be present.
ENTACTOGENS	MDMA						
	MDE				NO REACTION		
	MDA ("Sass")						
	5-APB / 6-APB						
CATHINONES ("BATH SALTS")	Methylone				NO REACTION		
	Ethylone				NO REACTION		
	Butylone			NO REACTION	UNKNOWN		
	Mephedrone (4-MMQ)	NO REACTION			UNKNOWN	UNKNOWN	
	MDPV			NO REACTION	UNKNOWN		
	Alpha-PVP ("Flakka")	NO REACTION	NO REACTION	NO REACTION		NO REACTION	
	Methamphetamine		NO REACTION			NO REACTION (BUT POSSIBLE RED)	UNKNOWN
	Amphetamine		NO REACTION			NO REACTION (BUT POSSIBLE RED)	
PIPERAZINES	BZP	NO REACTION	NO REACTION	NO REACTION		NO REACTION	
	TFMPP	NO REACTION	NO REACTION	NO REACTION		NO REACTION	NO REACTION
	PMA ⚠	NO REACTION	NO REACTION				
	PMMA ⚠	NO REACTION	NO REACTION				
	2C-B						
	2C-I			NO REACTION	UNKNOWN	UNKNOWN	UNKNOWN
	Ketamine	NO REACTION	NO REACTION		NO REACTION	NO REACTION	
	Methoxetamine	NO REACTION			NO REACTION	NO REACTION	
OPIATES	DXM			NO REACTION	UNKNOWN	NO REACTION	UNKNOWN
	Codeine				UNKNOWN		UNKNOWN
	Morphine			NO REACTION	UNKNOWN		
	Oxycodone				UNKNOWN		UNKNOWN
	Heroin				UNKNOWN		UNKNOWN
	Mescaline				NO REACTION		
	Cocaine	NO REACTION	NO REACTION		NO REACTION	UNKNOWN	
	Aspirin		NO REACTION		NO REACTION		
	Sugar				NO REACTION		

ABOUT THE KITS:

There are hundreds of drugs being sold as "Molly" or "Ecstasy." Whether you have pressed tablets, crystals or powder, there is no way to tell what drug you have without testing it first. We have tried to include the more common drugs on this color chart, but the situation is constantly changing. Visit www.dancesafe.org/testing-kit-instructions for the most up-to-date version.

Different reagents are better than others for identifying specific drugs. Our "standard set" provides the best overall sequence for identifying MDMA. MANDELIN is the best for ketamine. EHRLICH's (shown on the back page) is the best for LSD. FROEHDE and LIEBERMANN can help identify some of the common cathinones.

WARNING! Reagents can only detect the presence of a substance. They cannot detect strength or purity.

ABOUT THE LIQUIDS:

MARQUIS and MECKE begin as clear liquids and gradually turn dark over a number of months. Even when the liquid is very dark inside the bottle, the reagents will still work so long as you are able to see the color reactions on a white ceramic plate.

Fresh MANDELIN starts out light orange in color and after 3-4 weeks will turn yellow and start to separate inside the bottle. Make sure and shake it before each use. Mandelin has a shelf life of about six months.

FOLIN has a natural, light orange tint when both liquids (A and B) are combined. Be careful not to confuse this with a positive reaction. For comparing the color change use a "blank" next to your sample on the plate.

⚠ WARNING! PMA & PMMA ARE EXTREMELY DANGEROUS ADULTERANTS! THEY ARE NOT RECREATIONAL DRUGS AND HAVE CAUSED HUNDREDS OF ACCIDENTAL DEATHS.